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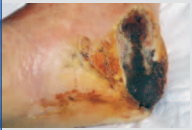
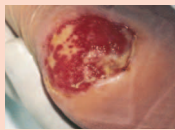
Wondreiniging			Granulatie	Epithelialisatie
Necrose	Infectie	Fibrinebeslag		
Oppervlakkige wonden				
				
<ul style="list-style-type: none"> ▶ TenderWet 24 active ▶ Hydrosorb gel 	Exsudaat ++ <ul style="list-style-type: none"> ▶ TenderWet 24 active ▶ Sorbalgon ▶ Atrauman Ag met absorberend verband 		Huid rondom de wond intact <ul style="list-style-type: none"> ▶ PermaFoam comfort ▶ TenderWet 24 active Huid rondom de wond beschadigd <ul style="list-style-type: none"> ▶ PermaFoam 	Huid rondom de wond intact <ul style="list-style-type: none"> ▶ Hydrocoll thin ▶ Hydrosorb comfort ▶ Hydrofilm Huid rondom de wond beschadigd <ul style="list-style-type: none"> ▶ Hydrosorb ▶ Hydrotul met absorberend verband
	Exsudaat + <ul style="list-style-type: none"> ▶ TenderWet 24 active ▶ Atrauman Ag met absorberend verband 		Huid rondom de wond intact <ul style="list-style-type: none"> ▶ TenderWet 24 active ▶ Hydrosorb gel 	
Diepe en/of kloofwonden				
				
<ul style="list-style-type: none"> ▶ TenderWet active cavity ▶ Hydrosorb gel 	Exsudaat ++ <ul style="list-style-type: none"> ▶ TenderWet active cavity ▶ Sorbalgon/Sorbalgon T ▶ Atrauman Ag met absorberend verband 		<ul style="list-style-type: none"> ▶ PermaFoam cavity ▶ Sorbalgon/Sorbalgon T ▶ TenderWet active cavity 	Huid rondom de wond intact <ul style="list-style-type: none"> ▶ Hydrocoll thin ▶ Hydrosorb comfort ▶ Hydrofilm Huid rondom de wond beschadigd <ul style="list-style-type: none"> ▶ Hydrosorb ▶ Hydrotul met absorberend verband
	Exsudaat + <ul style="list-style-type: none"> ▶ TenderWet active cavity ▶ Atrauman Ag met absorberend verband 		<ul style="list-style-type: none"> ▶ TenderWet active cavity ▶ Hydrosorb gel 	

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